

Cavex Alginate Mixer II

Automatic mixer for Alginate impression material

User manual

Installation – Operation – Troubleshooting

Contents

Page 2	General information for the user
Page 3	Description of the Alginate Mixer, with accessories
Page 4	Installation
Page 5	Operation
Page 8	Setting the optimum mixing time
Page 8	Maintenance
Page 9	Troubleshooting
Page 11	Specifications

General information for the user

Congratulations on the purchase of your Cavex Alginate Mixer II: an automatic mixer for alginate impression material!

Please read this user manual carefully before using the Alginate Mixer II. Its proper use will improve the quality of the mixed alginate impression material and also increase the mixer's working life. You should therefore keep the user manual in a safe place and ensure that employees who will be using the Alginate Mixer II receive proper training.

Important

Cavex Holland BV cannot accept responsibility for any inadequate performance, damage or accidents resulting from a failure to connect the Alginate Mixer II in the manner specified below; using the Alginate Mixer II for materials other than alginate impression material, or in a manner other than that described below; using parts other than those supplied for the intended purpose by Cavex, or repairs or modifications other than those made by experts appointed by Cavex for that purpose.



Description of the Alginate Mixer II, with accessories

- 1) Mains switch (I/O)
- 2) Power button (**Power**)
- 3) Memory button (**Memo/Mode**)
- 4) Start/stop button (**Start/Stop**)
- 5) Display
- 6) Buttons for setting the mixing time
- 7) Memory indicator (**Up/Down**)
- 8) Power lead
- 9) Power lead connector
- 10) Fuse holder
- 11) Mixing cup
- 12) Mixer case
- 13) Spatula
- 14) Stirrer
- 15) Dose-measuring jug
- 16) Plastic container (with magnet)

Installation

1. Remove the two locking screws underneath the Alginate Mixer II (*fig. 1*).

N.B. Keep the locking screws in a safe place. If the Alginate Mixer II is transported (e.g. if sent for repair) they will have to be screwed in again to prevent further damage.

2. Place the Alginate Mixer II on a solid surface that is stable and flat. Ambient temperature: 2°C – 40°C (35.6°F - 104°F).
3. Check that the voltage indicated on the device is the same as your mains supply voltage and connect the Alginate Mixer II to an earthed power point using the power lead provided (*fig. 2*).

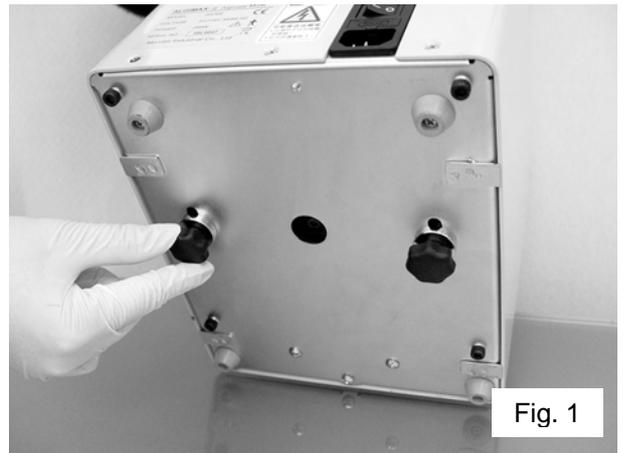


Fig. 1



Fig. 2

Operation

1. **Switch on the Alginate Mixer II**
Switch on the power using the mains switch (**I/O**) (fig. 3) on the back of the device and then press the power button (**Power**) (fig. 4) on the control panel. The entered mixing time in seconds will be indicated on the display. Choose the required mixing time using the memory button (**Memo/Mode**).



N.B. 1. The following mixing times have been preprogrammed at the factory:

Memory location	Mixing time, sec.
1	8
2	10
3	12

You can easily change the mixing times to suit your own requirements and store them in memory.

See: **Setting the optimum mixing time** on page 7.

N.B. 2. The recommended mixing times for CA37 Normal Set and Cavex Impressional Normal Set are between 8 and 12 seconds for 1, 2 and 3 small spoonfuls of powder. Extending the Alginate Mixer II mixing time will reduce the hardening time by a few seconds.

N.B. 3. In general, alginate impression material mixed in the Alginate Mixer II will harden sooner than you are used to when mixing by hand. This is because the Alginate Mixer II mixes more thoroughly. Using the normal setting for alginates is therefore recommended.



- Put the required amount of powder to the mixing cup (fig. 5) and add the required volume of water, according to the manufacturer's instructions.

For optimum mixing, mix slightly beforehand using the stirrer provided (fig. 6).

- N.B. 1.** The total contents of the mixing cup, powder + water, must not exceed 100 g.
- N.B. 2.** It is important to close the lid properly to prevent alginate impression material from leaking out during mixing. This could adversely affect the operation of the Alginate Mixer II.
- N.B. 3.** Replace the lid and/or mixing cup when it can no longer be closed properly owing to wear.
- N.B.4.** If required, the preparation proportions for the Cavex alginate can be adjusted. To this end, the volume of water can be reduced (by no more than 30% below the recommended volume). Excess water may reduce the mixing performance of the Alginate Mixer II and lead to lumps in the alginate impression material.





Fig. 7



Fig. 9

3. Close

Close the mixing cup with the lid. Press to close tightly (fig 7).

4. Mix

Place the filled and closed mixing cup in the holder (fig. 8). Close the cover of the Alginat Mixer II properly (otherwise the mixer will not start) and press the start/stop button (**Start/Stop**), as shown in (fig. 9). The mixer starts and the displayed mixing time will count down until the end of the mixing time, when the mixer will stop automatically.

N.B.1. You can stop the Alginat Mixer II at any time by pressing **Start/Stop**.

N.B.2. Only open the cover when the mixer has come to a halt.

5. The result

Open the cover when the mixer has completely stopped and remove the mixing cup from the holder. You can easily remove the mixed alginate impression material from the mixing cup using the spatula (fig. 10).



Fig. 10



Fig. 8

Setting the optimum mixing time

Set the optimum mixing time as follows:

- Use the memory button (**Memo/Mode**) to select the required memory location.
- Set the required mixing time (in sec) with the **Up** or **Down** button.
- Now press the **Up** and **Down** buttons simultaneously: you will hear 2 high-pitched sounds and the display will flash twice.
- The selected mixing time has been saved to memory.

Different mixing proportions

Using 30% less water than recommended produces a "stiff alginate". This is the ideal consistency for making the functional impression (initial impression for making a suitable individual mould) for a stable prosthesis.

Maintenance


1. Mixing cup

Thoroughly clean the mixing cup and lid after mixing. Hardened alginate impression material can be easily removed. Do not use a sharp object like a knife, as this will scratch the surface.

Ensure that the hole in the lid of the mixing cup does not become blocked (*fig. 11*), as that would make it extremely difficult to open and close the lid.

2. Mixer

Clean with a damp cloth, possibly using a neutral cleaning agent.
Never use solvents!

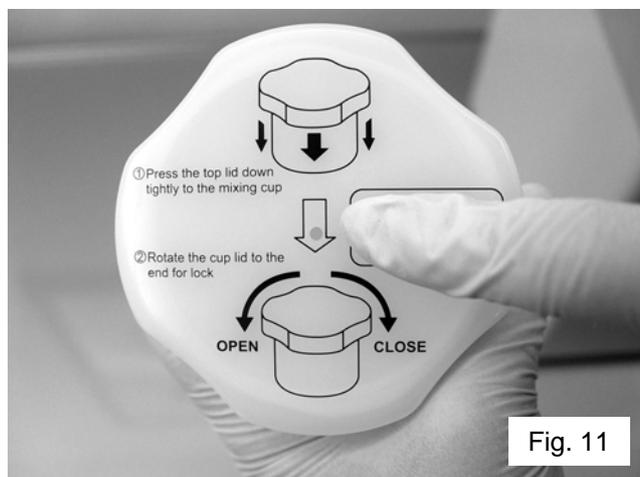

Troubleshooting

Problem	Cause	What to do
Appliance will not switch on.	Power lead is not connected.	Check the power lead connection to the appliance and power point.
	Fuse has blown	Check for external causes and correct them. Then replace the fuse. If the fuse blows again, send the appliance for repair*.
Display does not light.	Printed circuit board defective.	Send the appliance for repair*.
	Motor or drive defective.	Send the appliance for repair*.
Speed of rotation too low.	Motor or drive defective.	Send the appliance for repair*.
	Starter capacitor defective.	Send the appliance for repair*.
Abnormal noise or vibration while rotating.	Mixing cup too heavily laden.	Total weight of powder + water must not exceed 100 g.
	Mixing cup is empty.	Never switch on the Alginate Mixer II when the mixing cup is empty.
	Leakage of powder and/or water during mixing.	Always close the lid of the mixing cup properly. Watch out for damage to the cup or lid.
	Drive defective.	Send the appliance for repair*.

* see list of distributors

NB: Do not forget to put back the two locking screws before transporting the appliance for repair (see fig.1).

Specifications

Voltage	AC 220 – 230 V, 50/60 Hz
Power consumption	220 W
Speed of rotation	3400 rpm (60Hz), 2900 rpm (50Hz)
Timer	Adjustable in seconds from 1 – 16 sec
Memory locations	Three: factory default settings 8, 10 and 12 sec
Dimensions	wxdxh: 205x250x295 mm
Weight	16 kg (35.2 lb)

Cavex Holland BV
Fustweg 5
2031 CJ Haarlem
The Netherlands
T +31 23 530 77 00
Dental@cavex.nl
www.cavex.nl

03-2018