## CAVEX NON GAMMA-2

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### EN (English)

## CAVEX NON GAMMA-2

**Cavex Non Gamma-2** is a dispersed phase amalgam alloy: a mixture of lathe-cut particles and micro-fine esthetic spherical particles. The handling characteristics are comparable to a conventional amalgam: sufficient condensation pressure should be applied.

The amalgaam made of  **Cavex Non Gamma-2** is free from the corrosion-prone and weak gamma-2 phase. Durable restorations with a lasting lustre can be made.

Silver (Ag)	: 69,2% (m/m)
Tin (Sn)	: 18,6% (m/m)
Copper (Cu)	: 1,9% (m/m)
Zinc (Zn)	: 0,3% (m/m)

**Cavex Non Gamma-2** is in full compliance with the latest version of the ISO24234 and ADA No. 1 specifications. The mercury contained in the pre-dosed capsules is in full compliance with ISO24234.

#### Indication

Stress-bearing fillings in posterior teeth (Class I and II).

#### Contraindications

- Use of amalgam is not recommended when a physician has deemed it unsuitable for the patient.
- Si sconsiglia l'impiego dell'amalgama qualora un medico abbia constatato un'intolleranza da parte del paziente.
- Use of amalgam is not recommended for the treatment of deciduous teeth, children under 15 years and pregnant or breastfeeding women, except when strictly deemed necessary by the practitioner on the ground of specific medical needs of the patient (from 1 July 2018).
- Hypersensitivity (allergic reaction) to amalgam components.
- In such cases the use of suitable alternative materials must be considered.

#### Warnings

- After placement or removal of dental amalgam restorations increased mercury concentration in blood and urine has been observed. According to available scientific knowledge this increase has not been associated with any adverse health effects.
- If placed in close contact with other metal restorations, galvanic effects may occur. In most cases they will not persist for a long duration. If the effect persists, the user should consider replacement of the dental amalgam filling with another material.
- In individual cases, local mucosal reactions (lichenoid) have been observed. Such local reactions may be of an irritative (mechanical, chemical, electrochemical) or allergic nature. It is not clear whether these reactions are the result of the amalgam restoration (mechanical, electrochemical) or allergic in nature. In case of allergy to components of dental amalgam the use of suitable alternative materials must be considered.
- Unnecessary exposure to mercury vapour or dental amalgam particles during handling, placement or removal of dental amalgam should be avoided. Placement and removal of dental amalgam fillings should be performed with appropriate water spray and vacuum suction.
- Instructions and regulations for storage and disposal of dental amalgam waste must be observed.

#### Directions for use

**Mixing**  
 **Cavex Non Gamma-2** is available as pre-dosed capsules. It should be noted, that the working- and hardening time would be somewhat different, dependent on the type of mixer used: a high-energy mixer gives a faster hardening. The mixed amalgam should be a plastic, coherent mass with a shiny surface.

Mixing ratio for Cavex Non Gamma-2is: 10 parts alloy / 11 parts mercury by weight

**Cavex Non Gamma-2 in all cases 5 – 7 seconds in High-energy mixer (such as Silamat)**

- I-spill (alloy/mercury 400/435 mg)
- II-spill (alloy/mercury 600/653 mg)
- III-spill (alloy/mercury 800/865 mg)

**Note:**

- For safety reasons it is best to reclose the empty capsule after taking out the amalgam, before cleaning or disposal.
- The pre-dosed capsules (Type 1) are self-activating: contact between alloy and mercury occurs automatically when the capsule is subjected to the action of the mixing machine. Length: 32 mm, max. exterior diameter: 14 mm.
- Storage at temperatures no higher than 25°C is recommended.
- Dispose of cotton wool and used capsules in accordance with national regulations.

**Insertion and condensation**  
After mixing, the amalgam should immediately be inserted into the cavity using an amalgam carrier. The filling should be build up in layers under adequate condensation pressure using a pluggger appropriate for the size of the cavity. Do not use an ultrasonic condenser. Carefully remove excess mercury.

#### Finishing and polishing

Approx. 8-10 minutes after mixing, the filling can be finished. Polishing should be done in a subsequent session.

<p><b>"THIS ALLOY CONTAINS ZINC; THE AMALGAM MADE THEREFROM MAY SHOW EXCESSIVE EXPANSION IF MOISTURE IS INTRODUCED DURING MIXING OR COMPACTING"</b></p>
  

**Labelling**  
This information is in compliance with ISO24234 and also with European legislation.

<p><b>MERCURY DANGER</b></p> <ul style="list-style-type: none"><li>May impair fertility or cause harm to an unborn child.</li> <li>Lethal if inhaled.</li> <li>Causes damage to organs in case of long-term or repeated exposure.</li> <li>Highly toxic to aquatic organisms, may cause long-term adverse effects.</li></ul>
  

#### Recommendations for safety precautions

Dental clinical personnel, when using silver amalgam, should exercise proper procedures to avoid mercury exposure. Precautions should be taken to avoid exposure to mercury vapour in the dental environment. Waste amalgam should be collected for disposal in a manner, which will protect both those who handle the waste and the environment.

- Treat mercury and amalgam with care. Be MERCURY-AWARE.
- Ideally a high-energy mixer should be equipped with a protective cap to cover the capsule whilst mixing.
- Small droplets of spilled mercury can be collected with the aid of a mercury-collecting forceps' or a small amount of freshly mixed amalgam that will easily absorb liquid mercury. NEVER use a vacuum cleaner to remove spilled mercury.
- Placement and removal of amalgam fillings must be done under water-cooling and with effective vacuum suction. The use of eye protection and mouth mask is advisable.
- The wastewater drain of the spittoon should be equipped with an amalgam separator.
- Local regulations on type and installation of such a separator and handling/recycling of the amalgam waste should be taken into consideration.** Dispose of used capsules in a container, designed for the purpose, or at least in a closed container filled with photographic fixer.

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Our technical advice, whether verbal, in writing or by way of trials, is given in good faith but without warranty, and this also applies where proprietary rights of third parties are involved. It does not release you from the obligation to test the products supplied by us as to their suitability for the intended processes and uses. We do not accept any liability for damages, either directly or indirectly, arising from the use of our products, entirely your own responsibility. Should, in spite of this liability be established for any damage, it will be limited to the value of the goods delivered by us and used by you. We will, of course, provide consistent quality of our products within the scope of our General Conditions of Sale and Delivery.

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