
Cavex VacuFormer

Cavex Holland BV · Fustweg 5 · 2031 CJ Haarlem · The Netherlands
T +31 (0)23 530 7700 · dental@cavex.nl · www.cavex.nl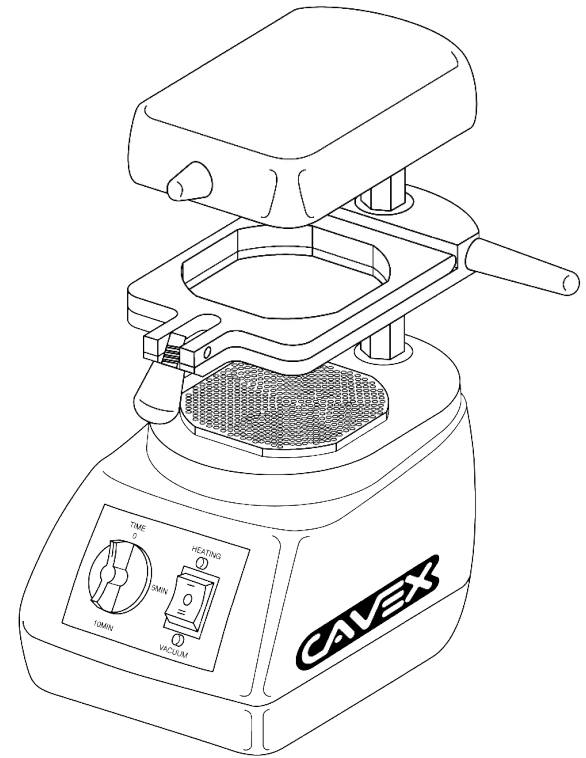

Content

Dutch (NLD)	1~9
English (GBR)	10~17
German (DEU)	18~25
Spanish (ESP)	26~33
French (FRA)	34~41

March 2016

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Contents

Technical information	10
Introduction	11
Parts	12
Instructions for use	13~15
Frequently asked questions / problems	16
Maintenance and troubleshooting	17

Technical information

Contents of box:

- 1 Cavex VacuFormer
- 1 Guarantee certificate
- 1 Connection lead
- 1 Manual

Technical specifications

Voltage:	AC230V/50Hz
Classification:	class 1
Model:	DV-1
Fuse:	250V8A
Vacuum power:	1000W
Heating power:	500W
Dimensions:	250 × 180 × 330mm
Net weight:	6,8 kg

Explanation of symbols

Hot

Note!

Earth

CE mark

Introduction

Dear new user,

Thank you for purchasing the Cavex VacuFormer (CVF) and congratulations on your choice.


Although the CVF is fairly easy to use, we strongly recommend you to read this manual carefully before you start!

To become familiar with using the different deep-draw variants (splint, whitening tray, dental protector, bite splint), we advise you to practise making a number of moulds based on the protocols described in this manual.

NOTE! Do not place the Cavex VacuFormer too close to walls, cupboards, other equipment, etc. Place the CVF on a stable surface with sufficient space around it.

Parts

1. Heating element
2. Frame for deep-draw sheet
3. Frame seal
4. Vacuum plate
5. Timer dial
6. Heating / Vacuum switch(*)
7. Height handle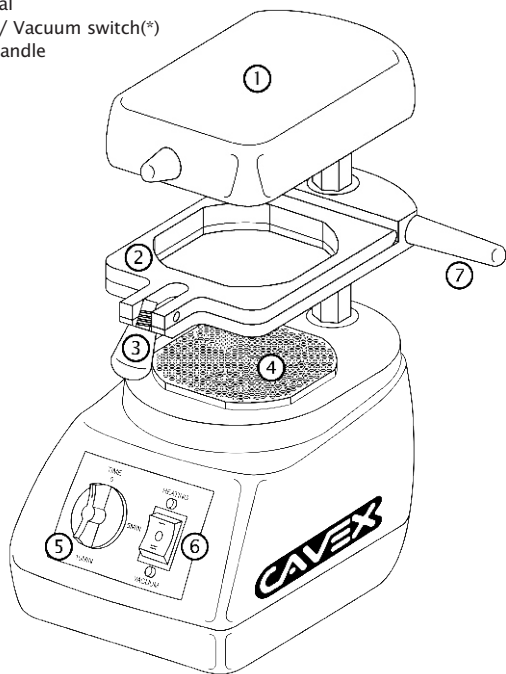

(*) Heating starts when the switch is set to "Heating" (-); the red light comes on.
The vacuum motor starts when the switch is set to "Vacuum" (=); the green light comes on.

Instructions for use

1. Swivel the heating element to the rear and press the Heating switch (fig. 1).
Preheating takes around 3 minutes.
Preheating is not always necessary but is always recommended for sheets thinner than 1 mm.

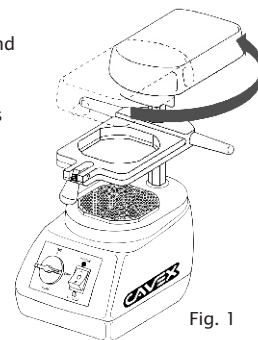
Fig. 1

2. Place the required deep-draw sheet in the frame (fig. 2) and close it carefully.
Slide the frame all the way up (fig. 3).

Fig. 2

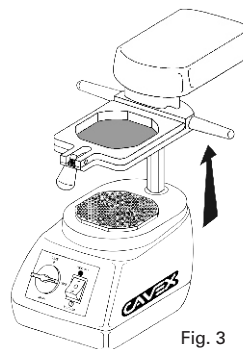
Fig. 3

Instructions for use

- Place the impression on the vacuum plate (fig. 4)

TiP: Ensure that the base of the impression is not too high. If it is, the sheet will be drawn occlusively too thin, which increases the likelihood of cracks in the incisal edges.

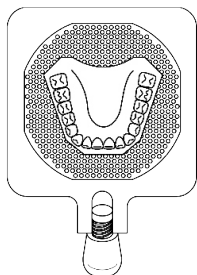
Fig. 4

- Swivel the heating element over the frame with the deep-draw sheet (fig. 5). Preheat the sheet until it hangs down by 2 to 3 centimetres (fig. 6)

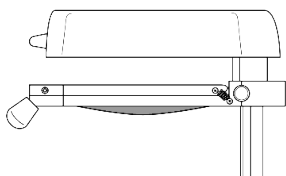
Fig. 6

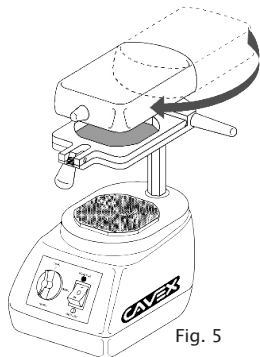
Fig. 5

Instructions for use

- Swivel the heating element to the rear again. Switch on the vacuum motor and use the handles (fig. 7) to pull the frame down over the impression. Allow the motor to run until the sheet has cooled.
- Remove the impression from the deep-draw sheet (fig. 8) and cut away the flat part of the sheet.
- Snip or cut the sheet as indicated to achieve the required result.

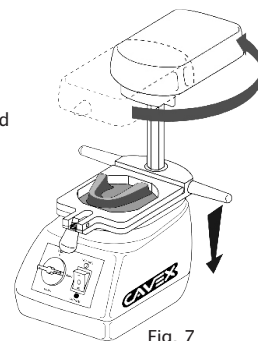
Fig. 7

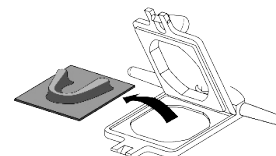
Fig. 8

NOTE!

- !! Only use earthed power outlets.
- !! For indoor use only.
- !! The CVF has overheating protection. The mechanism is automatically activated when the CVF is used for too long.
- !! Let the Cavex VacuFormer cool for at least 10 minutes after use. This prevents overheating.
- !! Do not leave the heating element over the handles for too long. The intense heat could melt them. The handles could also become hot enough to burn your hands.

Frequently asked questions / problems

I. The deep-draw sheet tears and/or stretches too much:

This can happen when the sheet has become too hot. The problem can be prevented by preheating sufficiently and swivelling the heating element to the rear in good time.

I. The sheet becomes extremely thin at the incisal edges:

This is often caused by a cast being too high. Trim the cast to a height of no more than 2.5 centimetres. This problem can be caused by the cast being too centralised. Place the anterior part closer to the centre.

I. Insufficient detail:

This is often caused by insufficient heating or a vacuum period that is too short.

I. The sheet gets stuck in the vacuum holes:

Overheating makes the sheet so soft that it is sucked into the vacuum holes during the vacuum process. Prevent overheating. Carefully try to extract the sheet from the holes.

Maintenance and troubleshooting

NOTE! *Never clean the Cavex VacuFormer with an organic solvent.*

After sales service

The Cavex VacuFormer is guaranteed for 1 year from the date of purchase. We are not liable for damage caused by unqualified persons.

Troubleshooting

Try the following if the CVF stops working properly. If this does not help, please contact the dealer.

Problem	Cause	Solution
There is no power when the machine is switched on.	The mains lead is not inserted into the power outlet properly.	Check the power supply.
	The fuse has blown.	Check the fuse.
The heating element does not work when the mains switch is switched to Heating.	The power supply is defective.	Return for repair.
	The overheating protection has been activated.	Wait for the element to cool and then try again.
	The timer has not been switched on.	Turn the timer dial to the required time setting.
The vacuum motor does not work when the switch is set to Vacuum.	The overheating protection has been activated.	Wait for the element to cool and then try again.
	The vacuum motor is defective.	Return for repair.
The frame does not move freely up and down.	The vertical shaft is dirty or needs more lubrication.	Clean and lubricate the vertical shaft.

Cavex Holland BV · Fustweg 5 · 2031 CJ Haarlem · The Netherlands
T +31 (0)23 530 7700 · dental@cavex.nl · www.cavex.nl

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